



HOW TO USE YOUR

GEL/FOAM CUSHION

Item #: 2601, 2601F-R, 2603C, 2603CF-R, 2647CG-R & 2604
Capacity not to exceed: **250 lb** (2604 models: **400 lb**)



**GREAT FOR
TRAVEL**

**ADDS COMFORT
WHILE SEATED**

HOW TO USE

Place bottom of cushion evenly on chair surface (some models have non-skid bottoms). For coccyx cushions (2603C, 2603CF-R, & 2647CG-R), place the U-shaped cutout at the rear of the seat such that it is directly under the tailbone area (**pictured**). When applicable, wrap velcro straps around back frame posts to secure cushion into position.

CARE AND MAINTENANCE

Check cushion before each use. Inspect cover for loose stitching, torn or frayed material. **DO NOT** use damaged cushions. Keep sharp objects away from cushion as they may puncture gel core.

To clean the cover:

See care label attached to cushion for washing instructions.

To clean the gel/foam core:

Remove the cover and spot clean with soap and water using a moist cloth. After cleaning, allow the foam to dry completely before replacing the cover and using.



WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA foam cushions are guaranteed to be free from manufacturing defects. NOVA will repair or replace the identified defect for a period of up to 12 months of use. In the unlikely event of a defect please call our customer service line listed below for instructions. Items not covered include normal wear and tear, foam or fabric discoloration, flattening, density variation, improper use, rental use, and neglect. Damage due to natural acts is not covered.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help

 info@novajoy.com
 1 800 557 6682

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!
novajoy.com
facebook: [novamedicalprod](#)
twitter: [novamedicalprod](#)
twitter: [novawalkerstalk](#)
YouTube: [NOVA Medical Products](#)

**Revised July 6, 2017*